


frida baby[®]

Electric NoseFrida[®] Pro

USER MANUAL

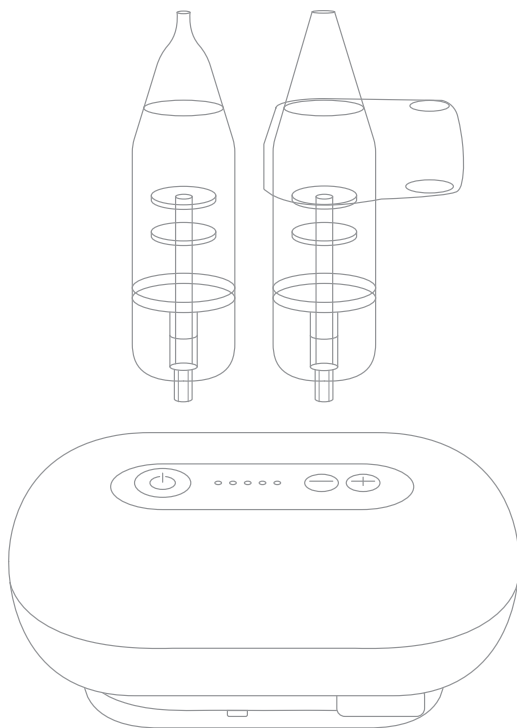

Easy Hold
Finger Grip

5



High Reach Soft
Tip Nozzle

7



6

Gentle Touch
Tip Nozzle



Soft Tip
Nozzle Body



Umbrella
(x2)




Nozzle Base
(x2)



Electric NoseFrida
Pro (Device)

1

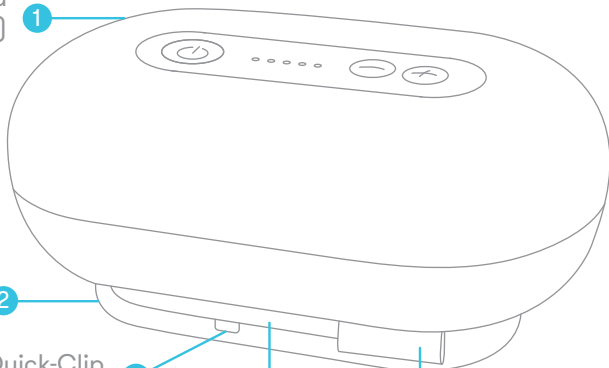

Base

2

Quick-Clip
Tubing
Connector

4

Tubing

3

Tubing Adapter

8

Product Specifications

KEY COMPONENTS

- 1 Electric NoseFrida Pro (Device)
- 2 Base
- 3 Tubing
- 4 Quick-Clip Tubing Connector
- 5 Easy Hold Finger Grip
- 6 Gentle Touch Tip Nozzle
 - Gentle Touch Tip/Body
 - Umbrella
 - Nozzle Base
- 7 High Reach Soft Tip Nozzle
 - Soft Tip Nozzle Body
 - Umbrella
 - Nozzle Base
- 8 Tubing Adapter
- Carrying Case
- USB-C Charging Cable

Specifications

- Product Name: Electric NoseFrida Pro
- Model Number: 12300100
- Input Power/Voltage: 5V=2A
- Product Dimensions: 140mm x 95mm x 65mm
- Typical service life of battery: 3 years
- ME equipment service life: 3 years
- Accessories service life: 3 years
- Applied part: High Reach Soft Tip Nozzle & Gentle Touch Tip Nozzle (Type BF)
- Vacuum pressure: 60-70 kPa
- Maximum flow value: 6.5 L/min
- Noise Level: ≤68 dB
- Power consumption: 5V 2A DC
- Motor Type: 5.0 V DC
- Weight: 0.77 lbs (350 g)

Tips Dimensions:

- High Reach Soft Tip: ID 2.4mm / OD 5.4mm
- Gentle Touch Tip: ID 3.7mm / OD 7.0mm

Storage Information

- Transportation Temperature: 32°F to 122°F (0-50°C)
- Humidity: 30% to 85% RH
- Pressure Altitude: 86 kPa to 106 kPa

Operating Conditions:

- Storage Temperature: 50°F to 95°F (10-35°C)
- Humidity: 30% to 85% RH
- Pressure Altitude: 86 kPa to 106 kPa

Indications for Use

The Frida Baby Electric NoseFrida Pro is intended for intermittent removal of nasal secretions and mucus in children. This device is intended for use in a home environment.

WARNINGS

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

DANGER – To reduce the risk of electrocution:

1. Do not use while bathing.
2. Do not place or store product where it can fall or be pulled into a tub or sink.
3. Do not place in or drop into water or other liquid.

WARNING – To reduce the risk of burns, electrocution, fire, or injury to persons:

1. Electric NoseFrida Pro should never be left unattended.
2. Close supervision is necessary when Electric NoseFrida Pro is used by, on, or near children or people with special needs.
3. Use this product only for its intended use as described in this manual.
Do not use attachments not recommended by the manufacturer.
4. Never operate the Electric NoseFrida Pro if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Never block the air openings of the Electric NoseFrida Pro or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
6. Never use while sleeping or drowsy.
7. Never drop or insert any object into any openings or hose.
8. Do not use outdoors or operate where aerosol (spray) products can be used or where oxygen is being administered.

WARNING:

- Only use for nasal aspiration; do not use on ears or throat.
- Only use as directed; misuse may lead to injury.
- When used on children, device should only be operated by a parent or adult caregiver.
- This is not a toy; keep out of the reach of children while not in use.
- Small parts are a choking hazard.
- Tubing is a strangulation hazard.
- Discontinue use in case of nosebleed or other nose injury.
- Discontinue use if there are signs of product damage or excessive wear.
- If symptoms do not improve, seek professional medical attention.
- Do not use the device in wet or humid environments.
- Do not attempt to open or repair device.
- Single user only. Do not use on more than one child.

SAVE THESE INSTRUCTIONS

CAUTION:

- Perform Regular Cleaning after each use as described in the Care + Cleaning section. If you observe snot in the tubing, follow the Deep Cleaning instructions.
- Only clean as directed. Do not use harsh chemicals as they may damage the device.
- Do not try to use this device while charging because it will not function.
- Do not apply excessive force when inserting nose tip in the nostril.

Power through sick days

When stubborn snot has your baby breathing like a freight train, level up your sick-day game with the Electric NoseFrida Pro. It puts our most effective SnotSucker® power at your fingertips, so you can vacuum boatloads of boogers at the push of a button. It's compact, portable, and easy to use with one hand, so you can cradle baby and conquer congestion without breaking a sweat.

Setup Instructions

Note: Clean and dry each piece of the nozzle prior to the first use.

Disassembly:

- Detach soft tip from nozzle body with a light pull.
- Detach umbrella from nozzle base with a light pull.
- Detach nozzle base from nozzle body with a light pull.


Assembly:

- Attach soft tip to nozzle body with a light push.
- Attach umbrella to nozzle base with a light push.
- Attach nozzle base to nozzle body with a light push.
- Reconnect device base by attaching and turning 90 degrees clockwise to secure.

Note: It is recommended to fully charge the Electric NoseFrida Pro prior to first use.

- Clean and dry the nozzle components (refer to Care + Cleaning Instructions).
- Ensure the nozzles are properly assembled.
- Remove tubing from Quick-Clip Tubing Connector and unwrap.
- Select nozzle and attach Easy Hold Finger Grip by sliding over the nozzle tip.
- Connect tubing to nozzle base by inserting tubing adapter into bottom of chosen nozzle.
- The device is now ready for use.

Operating Instructions

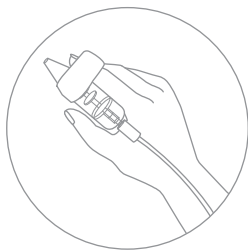

- Slide finger into Easy Hold Finger Grip for positioning and stability.
- Position your baby's head using your other hand.
 - Press the power button to turn on the device.
- **Caution – Device will not work while charging.**
- Use suction level adjusters (+/-) to choose your desired suction level.
 - Start on lowest setting and increase as necessary.
- **Note: Suction level is indicated by the 5 suction level indicators on the control panel.**
- Press the tip of the nozzle into your baby's nostril to create a seal.
- When powered on, The Electric NoseFrida Pro will remember the last used strength setting.
- Snot suck that baby!
- Repeat for other nostril if necessary.
- **Caution: If liquid is observed inside the tubing, discontinue use immediately. Clean and dry thoroughly prior to using again.**
- The unit has an Auto Shut-Off mode that automatically turns off the unit after 3 minutes.

Charging Instructions

- **Caution – Device will not work while charging.**
- Suction level indicators will flash red when the device is at low battery.
- To charge, plug USB-C cord into device. Plug other end of cord into USB wall charger (not included).
- Suction level indicators will blink blue as the device charges, filling from left to right to show charge status.
- When all five lights are solid blue, the device is fully charged.
- For a fully dead battery, charging to full can take up to 3 hours.
- Total device run time on a full charge is up to 2 hours.

Tips/Tricks:

- The device will remember your last suction strength setting.
- The device will not work while charging – make sure you plug it in after use to make sure you are always prepared!

Care + Cleaning


Regular Cleaning:

- Turn off device.
- Detach nozzle from tubing adapter.
- Disassemble the used nozzle completely.
- To clear snot off, wash the tip, nozzle body, nozzle base, and umbrella in warm water with mild soap.
- Fully rinse the tip, nozzle body, nozzle base, and umbrella with warm water.
- Let parts fully dry before reassembly.

NOTE: Perform Regular Cleaning after each use. If you observe snot in the tubing, follow the Deep Cleaning instructions.

Deep Cleaning: Snot should not enter the tubing during normal use. However, if snot is discovered in the tubing during inspection, the base of the device can be removed for additional cleaning.

- Turn base counterclockwise 90 degrees and pull to remove.
- Lightly pull tubing from tube connection point to disconnect.
- Wipe any snot from bottom of device.

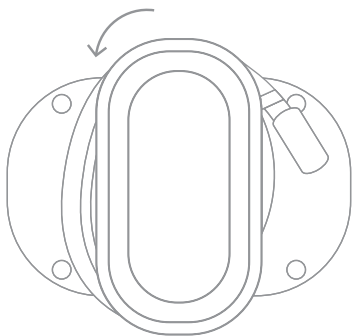

- **Caution - DO NOT ALLOW WATER TO ENTER INTO HOLES IN THE BOTTOM OF DEVICE.**
- Wipe base to remove all snot.
- Rinse tubing and base with warm water to fully clear the snot. Do not place either of these components in the dishwasher.
- Once disassembled, clean all components per the Regular Cleaning instructions above.
 - Optional: For easy cleaning, nozzle parts may be placed into a cleaning bag (not included) and washed on the top rack of the dishwasher on normal cycle only with dishwashing detergent known to be safe for household use.
- Let all parts fully dry before reassembling.
 - To reassemble the device, reattach tubing to tube connection point, insert base into device, and rotate 90 degrees clockwise to original position to fully secure.

Troubleshooting Guide

- **Device does not turn on:** Low battery – Plug the device in to make sure that it is charged. Device will not operate while charging.
- **Device is not charging:** Loose connection – Ensure that the charger is properly plugged in. When the device is plugged in and lights are flashing, the unit is charging. When the device is plugged in and lights are solid, the device is fully charged.
- **Suction is not working at all or is abnormally low:**
Low battery - Plug the device in to make sure that it is charged.
- **Dirty Parts:** Make sure that the tip, nozzle body & base, and umbrella are clean and that there are no obstructions.
- **Wet Parts:** Make sure that the tip, nozzle body & base, and umbrella are fully dry.
- **Loose Parts:** Make sure that the tip, nozzle body & base, umbrella, tubing, and base are assembled tightly together.
- **Device makes gurgle/murmur noise or debris starts to exit from the bottom of the device:** Snot may have entered the base of the Electric NoseFrida Pro. Please contact Frida customer support.

Made in China

FRIDABABY, FRIDA and all other trademarks herein are the registered and unregistered trademarks of Fridababy, LLC
© Fridababy, LLC 2025



Conforms to UL STD. 1431 & 2595

Certified to CSA STD. C22.2 NO. 68 & NO. 0.23

Manufactured for Fridababy, LLC.
82 NE 26th St. Miami, FL 33137 USA
fussbuster@fridababy.com

DISPOSAL INSTRUCTIONS

Make sure you dispose of, or recycle, your device in accordance with your local laws and regulations.

MRI Safety Information

The Frida Baby Electric NoseFrida Pro has not been evaluated for safety in the MR environment. It has not been tested for heating or unwanted movement in the MR environment. The safety of the Frida Baby Electric Nosefrida Pro in the MR environment is unknown. Performing an MR exam on a person who has this medical device may result in injury or device malfunction.

frida baby[®]

Love it? Leave a review.

fridababy.com

Electric NoseFrida Pro Questions?.

frida.com/help

**Manufactured for Fridababy LLC.
82 NE 26th St. Miami, FL 33137 USA**

123-LBL-0004V0