frida baby®

NoseFrida®

The OG SnotSucker®

It's smart. It's Swedish. It's maybe a little strange. It's the doctor-invented nasal aspirator that actually works!

To Use: Disposable filter should be in place to prevent mucus or bacterial transfer. Place large tube against child's nostril (not inside), creating a seal. Use red mouthpiece to suck out snot.

Tips: (1) Spray saline solution in each nostril to help loosen thick mucus before using NoseFrida. (2) Move tube in a circular motion against nostril—you will hear and see mucus coming out of nose. (3) Change filter after use to keep maintain suction.

To Clean: Dispose of filter. Wash large tube with soap and water. Clean thin tube with a few drops of rubbing alcohol. Firmly snap dry pieces together, inserting a new filter for next use.

Safety: Tube does not go inside the nose, so baby will not get injured if they move. Only use NoseFrida for nasal aspiration—do not use in ears or throat. DO NOT BLOW air into red mouthpiece while tube is placed at child's nostril. NoseFrida is NOT a toy—keep all parts away from children. Never attach NoseFrida to a machine.

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