

frida baby®

FeverFrida the Cool Pads

Instant cooling for fever discomfort

SIMPLE STEPS:

- Tear open pouch, trim pad if needed + peel off transparent film
- Apply gel side to child's forehead, back of neck, chest or other hot spot (on dry skin, away from hair)
- Discard after use (cooling lasts up to 8 hours)

Caution:

Do not apply to sensitive areas, damaged skin, near eyes and mouth. This product is for external use only. Only use under adult supervision. Parts may come off causing a choking hazard. Do not use past the expiration date. If high temperature persists, or if irritation or discomfort occurs, discontinue use and consult your pediatrician. Do not reuse.

Storage:

Store in a cool dry place. For extra cooling, store in the refrigerator (optional)

Manufactured for
Fridababy LLC
Miami, FL 33137

Made in China
Not for individual resale

Love it? Leave a review.
[fridababy.com](https://www.fridababy.com)

Cool Pads questions?
fussbuster@fridababy.com

LBL-41-002v1