

## RELAX, WE'LL SUCK IT FROM HERE

The cult-favorite NoseFrida® has a new power partner. With the push of a button, the Electric NoseFrida® safely vacuums out stubborn snot in a flash, no lung power required.

The one handed easy-grip and distraction light make it easier to wrangle your tantrum-y toddler, plus constant power means you can get the job done on-the-double so that your little one can breathe easier.

Manufactured for  
Fridababy LLC, Miami, FL 33137

Love it? Leave a review [fridababy.com](https://www.fridababy.com)  
Snotsucking questions? [fussbuster@fridababy.com](mailto:fussbuster@fridababy.com)

## SAVE THESE INSTRUCTIONS IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

**WARNING** - To reduce the risk of burns, fire, or injury to persons:

Do not use while bathing.

Do not place or store product where it can fall or be pulled into a tub or sink.

Do not place in or drop into water or other liquid.

Do not reach for a product that has fallen into water. Unplug immediately.

Close supervision is necessary when this product is used by, on, or near children or those that are too weak or sick to care for themselves.

Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

Never operate this product if it has a damaged cord or plug. If it is not working properly, if it has been dropped or damaged, or dropped into water, return the product to Frida for repair.

Keep the cord away from heated surfaces.

Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and similar materials.

Never use while drowsy or sleeping.

Never drop or insert any object into any opening or hose.

Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.

Recharge only with the charger specified in the manual. A charger that is suitable for one type of battery pack may create a risk of fire when used with another battery pack.

Use appliances only with specifically designed battery packs. Use of any other battery packs may create a risk of injury and fire.

When battery pack is not in use, keep it away from other metal objects, like paper clips, coins, keys, nails, screws or other small metal objects that can make a connection from one terminal to another. Shorting the battery terminals together may cause burns or a fire.

Under abusive conditions, liquid may be ejected from the battery; avoid contact. If contact accidentally occurs, flush with water. If liquid contacts eyes, additionally seek medical help. Liquid ejected from the battery may cause irritation or burns.

Do not use a battery pack or appliance that is damaged or modified. Damaged or modified batteries may exhibit unpredictable behavior resulting in fire, explosion or risk of injury.

Do not expose a battery pack or appliance to fire or excessive temperature. Exposure to fire or temperature above 265 F (130 C) may cause explosion.

Follow all charging instructions and do not charge the battery pack or appliance outside of the temperature range specified in the instructions. Charging improperly or at temperatures outside of the specified range may damage the battery and increase the risk of fire.

If the Electric NoseFrida is not operating, return to Frida for servicing to be performed by a qualified repair person using only identical replacement parts. This will ensure the safety of the product is maintained.

Do not modify or attempt to repair the Electric NoseFrida except as indicated in the instructions for use and care.

Use only with the listed/certified power supply. Never use unapproved sources.

Not to cure or prevent any disease. Consult a doctor before using.

Keep out of reach of children under 12 years. Only for use by an adult on a child.

Never use the Electric NoseFrida for purposes other than those it has been intended for.

Please follow the operating instructions for proper use.

Never submerge the Electric NoseFrida into water or put under running water.

Follow the "Care and Cleaning" instructions for proper cleaning methods.

Clean the unit after each use.

Only insert the tip enough to create a gentle seal around the edge of the nostril.

Do not over-insert.

Use the Frida brand provided tips. Do not use tips from a different manufacturer.

If irritation occurs discontinue use immediately.

Not serviceable. Do not open the housing.

Inspect the Electric NoseFrida before each use.

Discard at the first sign of malfunction. Do not use if it is overheating, operating irregularly, or is showing signs of damage to the housing.

## Contents

1 Electronic Nasal Aspirator, 1 Snot Container, 2 Silicone Tips, 1 USB cable, 1 User Manual

## Specifications & Technical Data

### Product Name:

Electric NoseFrida®

**Model:** 5005-A

**Battery Voltage:** 3.7V

**Charging Voltage:** 5V

**Input Amperage:** 1 A

**Power Source:** USB-C

**Battery Life:** 80 minutes,  
when fully charged

**Battery Size:** 950 mAh  
(battery not replaceable)


**Battery Type:** Li-Ion

**Weight:** 200g (with battery)

**Dimensions:** 115x46x32mm

**Storage/Using/Charging**

**Conditions:** 10-40°C

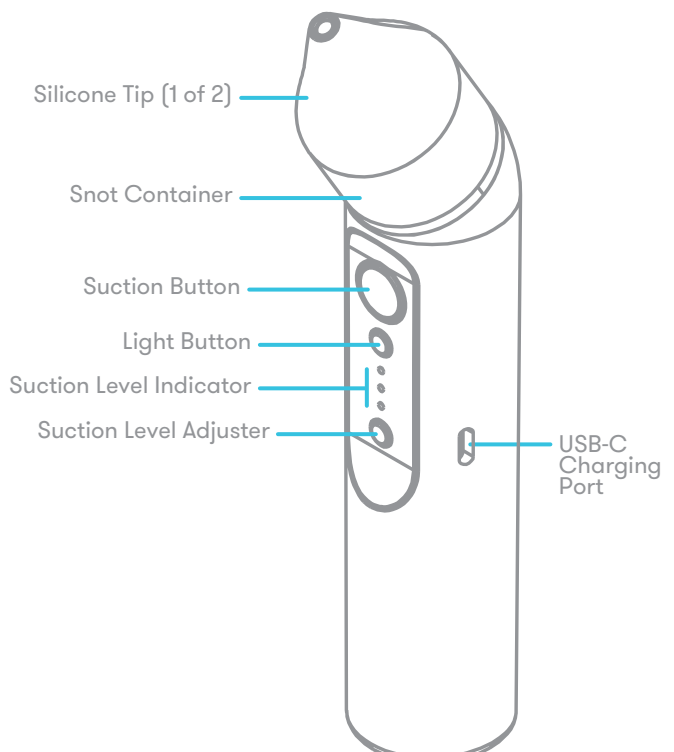

## FCC Statement

IMPORTANT INFORMATION REQUIRED BY THE FCC

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference; and (2) this device must accept any interference received, including interference that may cause undesirable operation.


## FCC Statement Cont.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: 1) Reorient or relocate the receiving antenna. 2) Increase the separation between the equipment and receiver. 3) Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. 4) Consult the dealer or an experienced radio/TV technician for help.


CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

## Operating Instructions

1. Attach the snot container.

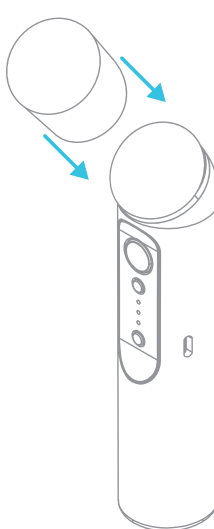

2. Attach the silicone tip. Pick the size that fits your child's nose the best.

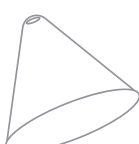


The NoseFrida tip is best for newborns up to age 2.



The extended tip is best for age 2 years +

3. Press the Suction Button 1x to turn the Electric NoseFrida on.

4. Press and hold the Suction Button to activate the suction. Keep pressing the button or the suction will stop.

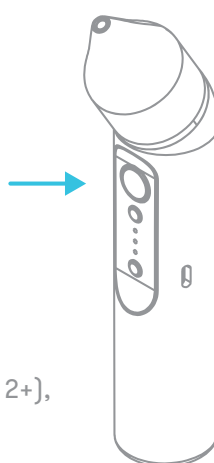

5. Place the NoseFrida tip gently against the child's nostril (not inside), creating a seal. If using the extended tip (age 2+), gently insert the silicone tip inside the nose.

6. Watch the snot container fill with gunk. Do not exceed the max line in the snot container.

7. Release the Suction Button for suction to stop.

8. The unit will shut off on its own after 3 minutes of no use.

### SUCTION LEVEL SELECTION:

Our Electric NoseFrida defaults to level 1 when it's turned on (level 1 LED will be lit).

To change to levels 2 and 3:

1. Press the Suction Level Adjuster button 1x to select level 2 (level 2 LED will light up).
2. Press the button 2x for level 3 (level 3 LED will light up).
3. Press the button 3x to return to level 1 (level 1 LED will light up again).

Test the suction level against the back of your hand to figure out which level to use before using it on your child.

### LIGHT BUTTON:

Nothing like a bit of distraction for a squirmy kid, amirite? If you want to use this feature, do it after turning on the power and before selecting the suction level.

1. Press the Light button 1x to turn on the rotating colors.
2. Press the button again to turn the light off.

## Care and Cleaning

After **each** use, remove the silicone tip, snot container and rinse with soapy water or toss them in the dishwasher. Clean the handle with a damp cloth.

Ensure that no liquid enters the interior of the Electronic NoseFrida. Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the instrument in water or other cleaning liquids.

## Trouble Shooting Guide

Check the following if your Electric NoseFrida is not working correctly:

Unit does not turn on.	Unit is not charging.	Suction is not working at all or is too low.
Low battery - Plug the unit in to make sure that it is charged.	Loose connection - Ensure that the charger is properly plugged in.	Low battery - Plug the unit in to make sure that it is charged. Dirty parts - Make sure that the silicone tip and chamber are clean and that there is no obstruction. Wet parts - Make sure that the tip and chamber are fully dry. Loose parts - Make sure that the tip and chamber are assembled tightly together.

# fridababy®

Love it? Leave a review [fridababy.com](http://fridababy.com)  
NOT IN LOVE? WE'VE GOT YOU COVERED.  
[fussbuster@fridababy.com](mailto:fussbuster@fridababy.com)  
1-833-WTF-FRIDA (1-833-983-3743)

Since we offer a lifetime warranty and will exchange a faulty unit on our dime, always consider dropping us a line before making a return.

Manufactured for Fridababy LLC, Miami, FL 33137